



318 mm

1. Gap between foot board and end of side rail must be $\geq 318\text{mm}$ or $\leq 60\text{mm}$

60 mm

4. Gap between head board and end of side rail must be $\leq 60\text{mm}$. Also applies to smallest gap from side rail to mattress platform.

Rail Ruler Guide



Yes, you can.[®]

safe patient handling

2. Gap between side rail and mattress platform must be $\leq 120\text{mm}$. Also applies to gaps in the side rails.

120 mm


Please see reverse for a diagram showing all four measurements and where they are applicable on a bed.

220 mm

3. Distance from top of mattress to top of side rails must be $\geq 220\text{mm}$

BS EN 60601-2-52

Bed Standard - Rail Ruler Guide

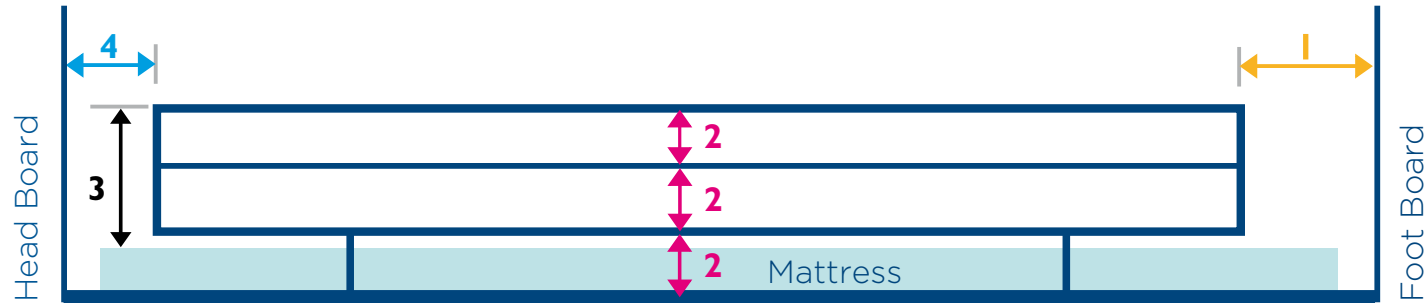

Please turn over to see what measurements apply to each number.

www.invacare.co.uk

Trusted Solutions,
Passionate People



Yes, you can.®

Invacare Ltd
Unit 4, Pencoed Technology Park
Bridgend CF35 5AQ
tel:01656 776222
email: uk@invacare.com

Invacare Ireland Ltd
Unit 5 - Seatown Business Campus
Seatown Road, SWORDS
Dublin
tel: +353 1 8107084
email: ireland@invacare.com